




Pupillary Distance RULER (PD RULER)

How to Measure Pupillary Distance Accurately

1. Print this ruler without scaling on your printer (double-check the measurements using a ruler).
2. Fold along the fold line.
3. Get a friend to help you for an accurate measurements, or use a mirror (see below in the ruler).
4. Place the ruler over the bridge of your nose. The 0 should be over your left pupil.
5. **LOOK STRAIGHT AHEAD.** On the ruler, mark what number lines up with the center of your right pupil.
6. **That's it!!!** Use this for ordering on the Average Joe's Eyewear Website.

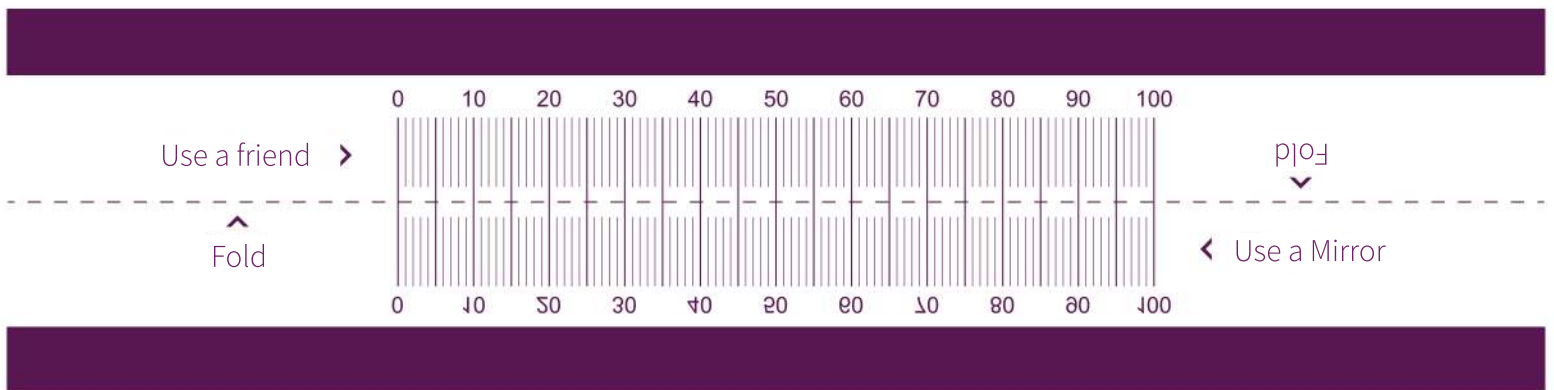

Measure against a ruler to assure printout is accurate.

